

UVGI Machine Decontamination of N95 Masks

Before the Call

After hand hygiene, retrieve two new N95 masks from stock. On exterior of masks clearly write your last name and ID number with a Sharpie.

Once labeled, place both masks in white paper bag on apparatus.

Procure new brown paper bag—write your last name and ID number on the exterior of the bag.

Use N95 mask per policy.

After the Call

Properly doff N95 mask, visually inspecting for soiling, saturation, seal and structural integrity.

If N95 passes inspection, carefully place in brown paper bag for UVGI decontamination. If not, discard.

Handle brown paper bags with gloves . Place brown paper bags in ventilated, plastic, secured bin.

Upon return to station, place ventilated, plastic, secured bin with brown bag(s) at designated UVGI N95 mask decon collection point.

Process used N95 masks in UVGI Machine per manufacturer's instructions.

Place a tally mark on N95 between straps with a Sharpie as they are unloaded. (Max of 4 marks)

Place N95 in white paper bag with name and ID number.

Restock decontaminated masks at designated station location.